


OsteoTape® Clinical Steps

Read product insert and related
protocols for tooth extractions

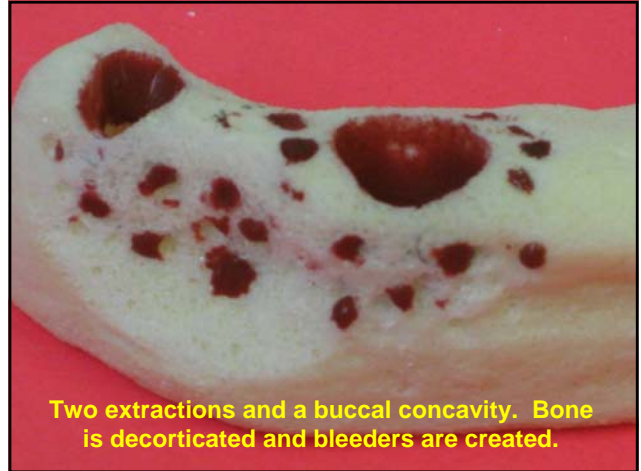



Two extractions and a buccal concavity. Bone is decorticated and bleeders are created.






Measure 14.0 mm strip for socket covering.






Mark measurement on top of primary bag.

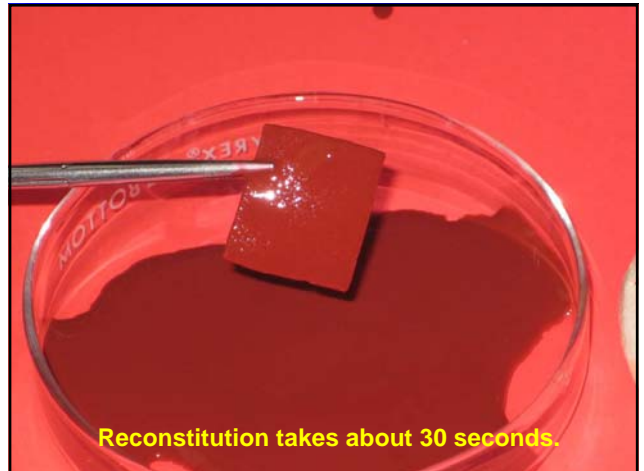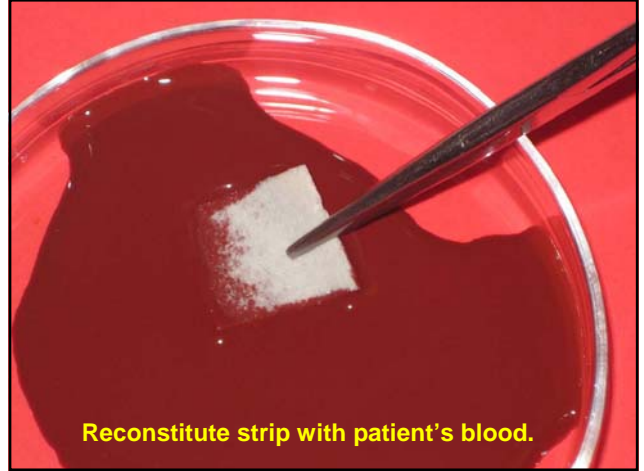

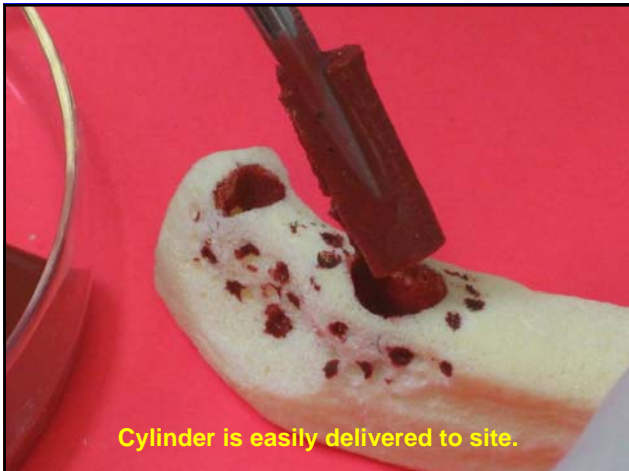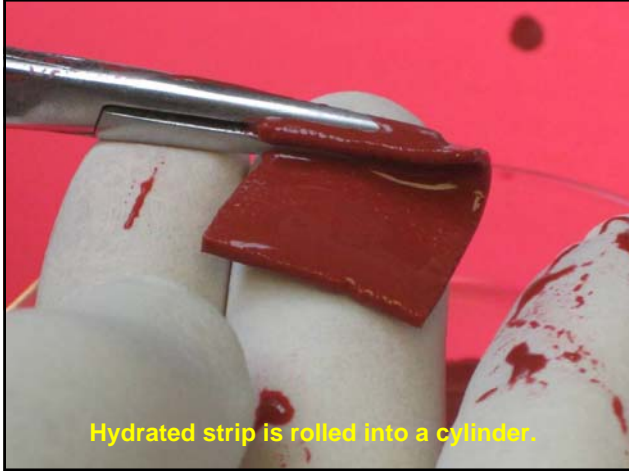

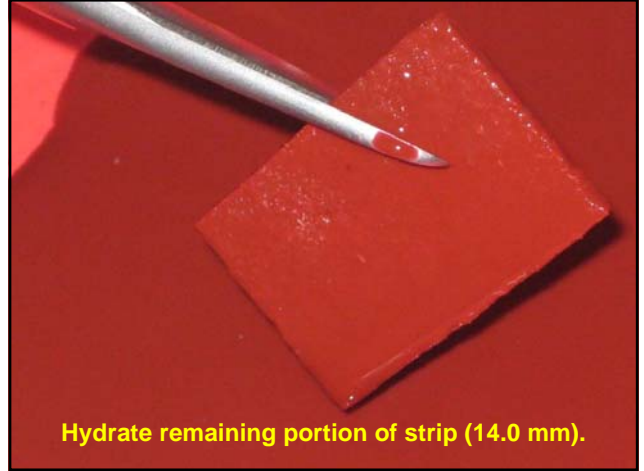

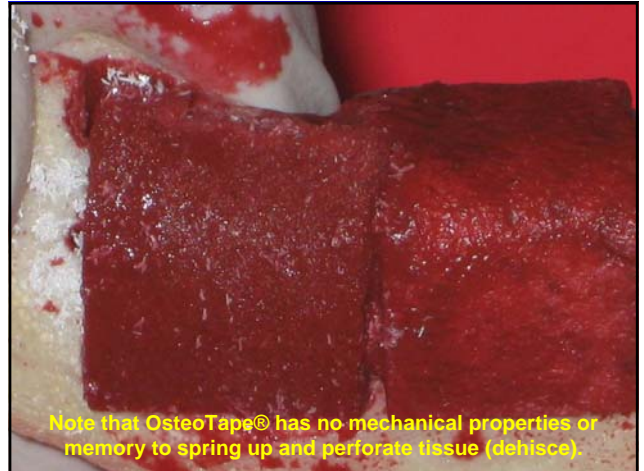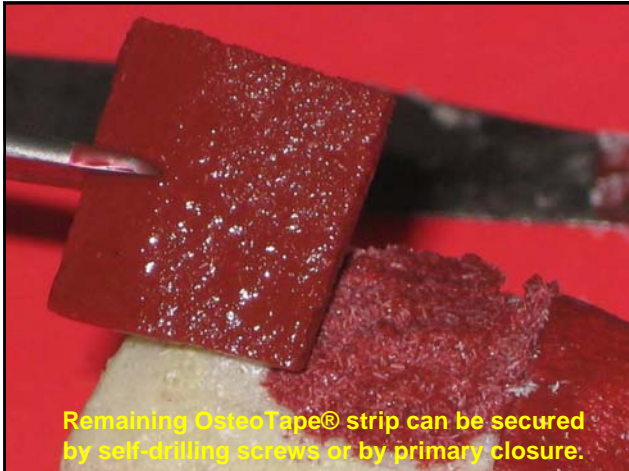

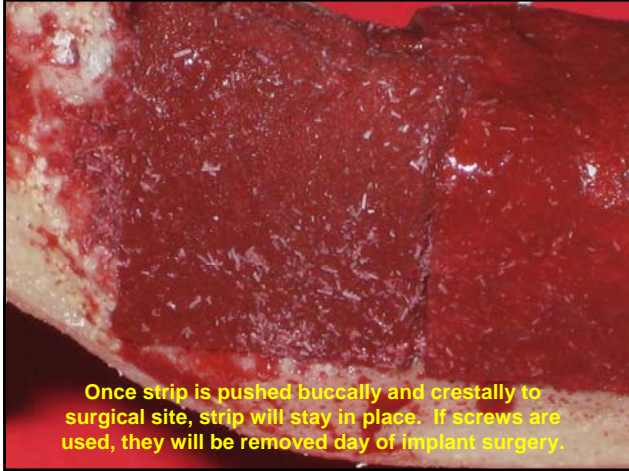

Do not load augmented area with prostheses.

OSTEOTAPE[®]
OSTEOGEN[®] WITH COLLAGEN RESORBABLE BONE GRAFT MATRIX

USE MEDULLARY BLOOD
 TO BUILD BETTER BONE

FAST AND EFFECTIVE BONE GRAFTING FOR PERIODONTAL INFRABONY DEFECTS AND IMPLANTS

EASILY MAINTAINED AT DEFECT SITES

RADIOLOUCENT DAY OF SURGERY
 NEW BONE WILL SHOW RADIOPAQUE

EACH LOT CERTIFIED NON-PYROGENIC

EXCELLENT HANDLING CHARACTERISTICS
 WHEN HYDRATED WITH MEDULLARY BLOOD

STRIPS AND GRANULES

SIMULATES NATURAL COLLAGEN MATRIX AND MINERAL STRUCTURE OF HUMAN BONE

DOUBLE-POUCHED AND STERILE IN A DURABLE BOX FOR PATIENT'S SAFETY

WWW.IMPLADENTID.COM
 800.536.9343